



Hy Laboratories Ltd.

6 Menachem Plaut St., Park Tamar, Rehovot 7670606, Israel

Tel. +972.8.9366475 Fax. +972.8.9366474

hylabs@hylabs.co.il | www.hylabs.co.il

Cat No. PD102 - HAEMOPHYLUS ANTIBIOGRAMMA AGAR (HAA)

Product Type: PETRI DISHES 90mm

Intended Use: medium formulated for the susceptibility testing of *Haemophilus* species isolated from clinical samples.

Principles and uses: The genus *Haemophilus* is comprised of a wide variety of clinically relevant species capable of causing infection in humans¹. These organisms are Gram-negative, film-forming, coccobacilli bacteria with marked pleomorphism and an ideal growth temperature of 35 to 37°C¹. HAA medium, based on the formulation developed by Jogensen, is recommended by the United States CLSI. The medium consists of Mueller-Hinton Agar with added yeast extract and supplemented with NAD (nicotinamide adenine dinucleotide) factor required for the growth of *Haemophilus* species. Yeast extract provides an additional source of growth factors. Beef extract and casein hydrolysate supply amino acids, nitrogenous substances, vitamins, and minerals necessary for growth. Agar is the solidifying agent. This medium was specially formulated for susceptibility testing of *Haemophilus influenzae*. HAA gives results that have been found to be highly reproducible. The transparency of the medium allows zones of inhibition to be read easily through the bottom of the plate.

Reference

1. Baron, S. 1996. Medical Microbiology. 4th edition. Galveston, Texas, USA.

Composition

Mueller Hinton Agar 38.0 g/L

Mueller Hinton Agar Typical Formula:

Dehydrated infusion from beef 300.0 g/L

Casein hydrolysate 17.5 g/L

Starch 1.5 g/L

Agar 17.0 g/L

Yeast Extract 5.0 g/L

NAD - nicotinamide adenine dinucleotide 16 mg/L

Growth factors 30 ml/L

Storage: 2-8 °C

Appearance: Light amber

pH Range: 7.2 - 7.4

Warning and Precautions - For professional use only. Follow good microbiological lab practices while handling specimens and culture. Do not use Petri dishes if they show evidence of microbial contamination, discoloration, drying, cracking, or other signs of deterioration. Avoid freezing and overheating. The Petri Dishes may be used / inoculated up to the expiration date and incubated for the recommended incubation times. After use and prior to discarding, specimen containers and all contaminated material, including the used culture media and contaminated culture containers, must be sterilized or incinerated by validated procedures. Since the nutritional requirements of organisms vary, some strains may be encountered that fail to grow or grow poorly on this medium.

If excessive moisture is observed, invert the bottom over an off-set lid and allow to air dry in order to prevent formation of a seal between the top and bottom of the plate during incubation. Storage Instructions: On receipt, store plates in the dark at 2-8 °C. Avoid freezing and overheating. Do not open until ready to use.

Performance Testing Results

GPT: Inoculum 10-100 cfu

TEST	ATCC	Incubation Temp. (°C)	Incubation Cond	Reaction	Antibiotic 1	Antibiotic 2	Antibiotic 3	Antibiotic 4
<i>Haemophilus influenzae</i>	49247	33-37 °C	5% CO ₂ , 18 hours	Sensitivity	AMP	CHL	IMP	SXT

AMP – Ampicillin 10ug, CHL – Chloramphenicol 30 ug, IMP - Imipenem 10ug, STX – Trimethoprim-1.25 ug /Sulphamethoxazole – 23.75ug.